




Product Reference Guide


Jelco™ ViaValve™ Safety IV Catheter

Guide for Successful Insertion


Insertion Technique

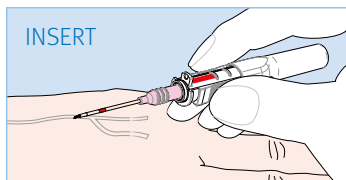


- › Apply tourniquet and prepare site according to policy.
- › Hold catheter by ribbed needle housing with thumb and fingers to insert needle into skin.
- › Visually inspect to confirm that needle bevel and push-off tab are facing up.
- › Anchor vein with gentle skin traction.
- › Insert needle at appropriate angle.

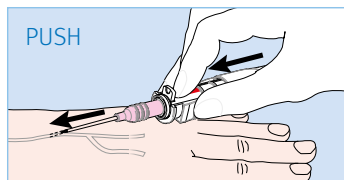

- › Observe for flashback.
- › If needed, slightly advance catheter and needle together to achieve full catheter entry into vein lumen.
- › Place index finger behind the primary push-off tab and PUSH catheter to thread to desired length.
- › **Do not reinsert needle into catheter at any time.**

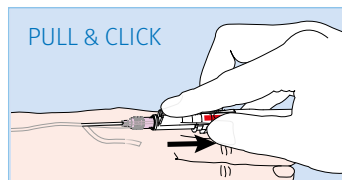

- › Stabilize device at push-off tab with index finger.
- › Holding ribbed needle housing, **PULL** needle into needle guard until you hear a **CLICK**.
- › **The click and visual inspection indicate that safety device has engaged successfully.**

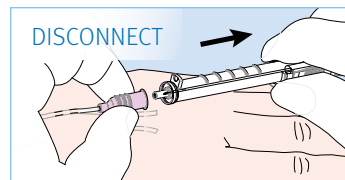

- › Remove tourniquet.
- › Apply digital pressure as needed beyond catheter tip.
- › Hold catheter hub and needle housing.
- › Disconnect needle housing by pulling backwards with a slight upward motion.
- › Connect Luer lock or tubing to hub per manufacturer's recommendation.
- › Secure connection with firm push and twist.
- › Stabilize and dress according to policy.