









# Product Reference Guide

## Safe-T-Lance® Plus Safety Lancet — Push Button Activated





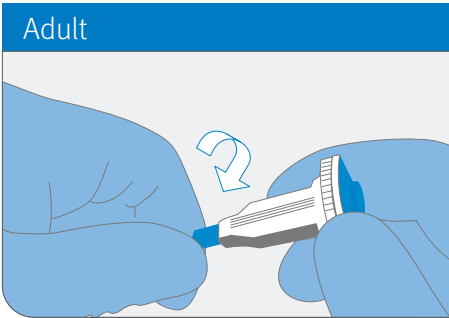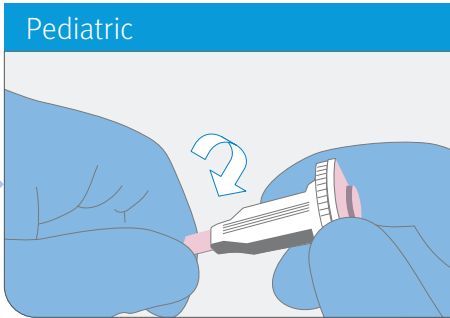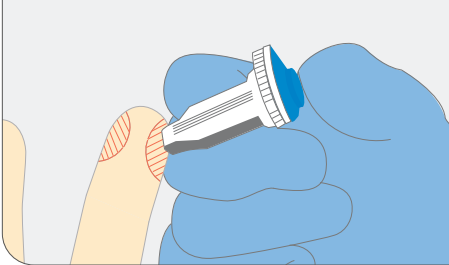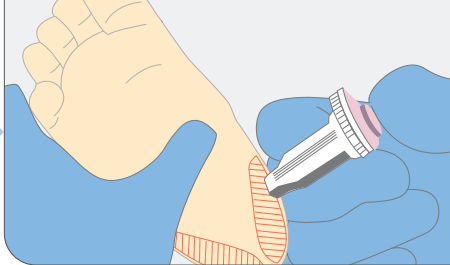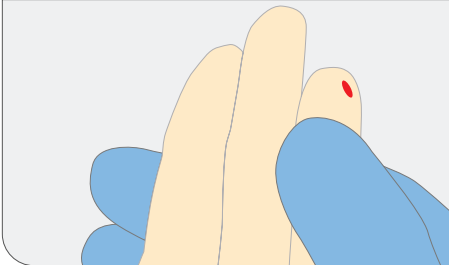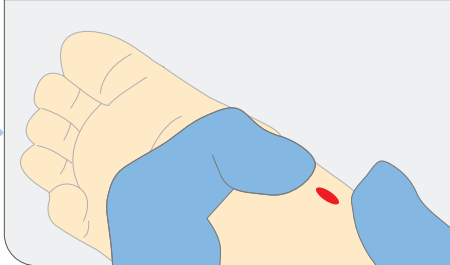

### Quick Reference Guide

Wash hands with warm water and soap. Dry thoroughly. Disinfect using an alcohol prep pad, ensure the finger (adult) or heel (infants/peds) is completely dry prior to lancing. Healthcare professionals, wear gloves prior to disinfecting the lancing site and completing the lancing procedure following the facility's protocol.

Adult		Pediatric
	<div style="background-color: #0070C0; color: white; width: 20px; height: 20px; margin: 0 auto; border-radius: 50%; display: flex; align-items: center; justify-content: center;">1</div> <p>Twist off the protective cap and pull the protective cap straight out.</p>	
	<div style="background-color: #0070C0; color: white; width: 20px; height: 20px; margin: 0 auto; border-radius: 50%; display: flex; align-items: center; justify-content: center;">2</div> <p>Press the safety lancet body firmly against the puncture site and push the button to activate the device. For adults, avoid the finger pads, lancing at the side or top of the finger.</p>	
	<div style="background-color: #0070C0; color: white; width: 20px; height: 20px; margin: 0 auto; border-radius: 50%; display: flex; align-items: center; justify-content: center;">3</div> <p>Gently apply intermittent pressure near the puncture site to obtain the required blood volume.</p> <p>Dispose of the entire safety lancet after use into an approved sharps container in accordance with facility guidelines and local regulations.</p> <p>The user should wash hands with warm water and soap after safe disposal of the safety lancet.</p>	

Use only for drawing capillary blood samples

Ordering Quantity: 200 Units/Box, 2000 Units/Case

Adult				Pediatric	
					
High Flow 9660	Normal Flow 9658	Low Flow 9659	Micro Flow 9663	Pediatric 9662	Max Flow 9661
18 gauge needle	21 gauge needle	25 gauge needle	28 gauge needle	1.5 mm blade width	1.5 mm blade width
1.8 mm depth	1.8 mm depth	1.4 mm depth	1.6 mm depth	1.2 mm depth	1.6 mm depth

**WARNING:** Do not use if protective cap has been previously removed from lancet. Clean the skin according to your safety procedures. Discard used lancet into a sharps container in accordance with facility guidelines, national and state regulations.

Safe-T-Lance is registered trademark of "HTL-Strefa" Spolka Akcyjna  
**Rx Only** For safe and proper use, refer to the appropriate manual, guide, or instructions for use.  
 Figures are rendered as graphic representations to approximate actual product.

