

Reference Guide for

NanoClave™ Manifold

Multi-port Infusion System



Disinfect to Protect

- › When attaching a new NanoClave manifold, prime each connector side port and the main line of the set by attaching a desired priming solution or normal saline, in accordance with facility protocol.
- › Before accessing a NanoClave, always disinfect the injection site with the approved antiseptic per facility protocol.
- › Scrub the injection site in accordance with facility protocol for appropriate scrubbing and dry times.



Administer

- › Attach IV tubing or syringe to NanoClave by inserting the luer and twisting until a friction fit is achieved.
- › Do not over-tighten a luer beyond the friction fit as this may damage both the luer and the NanoClave.



Flush After Each Use

- › Flush the manifold NanoClave connector per facility protocol. After blood use, the NanoClave can be flushed clean and does not require change-out.
- › Use routine flushing in accordance with facility protocol in order to maintain catheter patency.
- › Change NanoClave in accordance with facility protocol.



NanoClave's saline flush option is designed to help reduce the risk of Heparin Induced Thrombocytopenia (HIT).

icumedical
human connections

The product complies with current legislation and has the corresponding CE marking. For additional information, warnings and /or safety precautions, refer to the manufacturer's Instructions for Use.